


## Easy Implant Removal

# EFR KIT


Bone loss-less fixture removal  
Quick & Simple removal by two(2) Stage  
Above 80% removal success

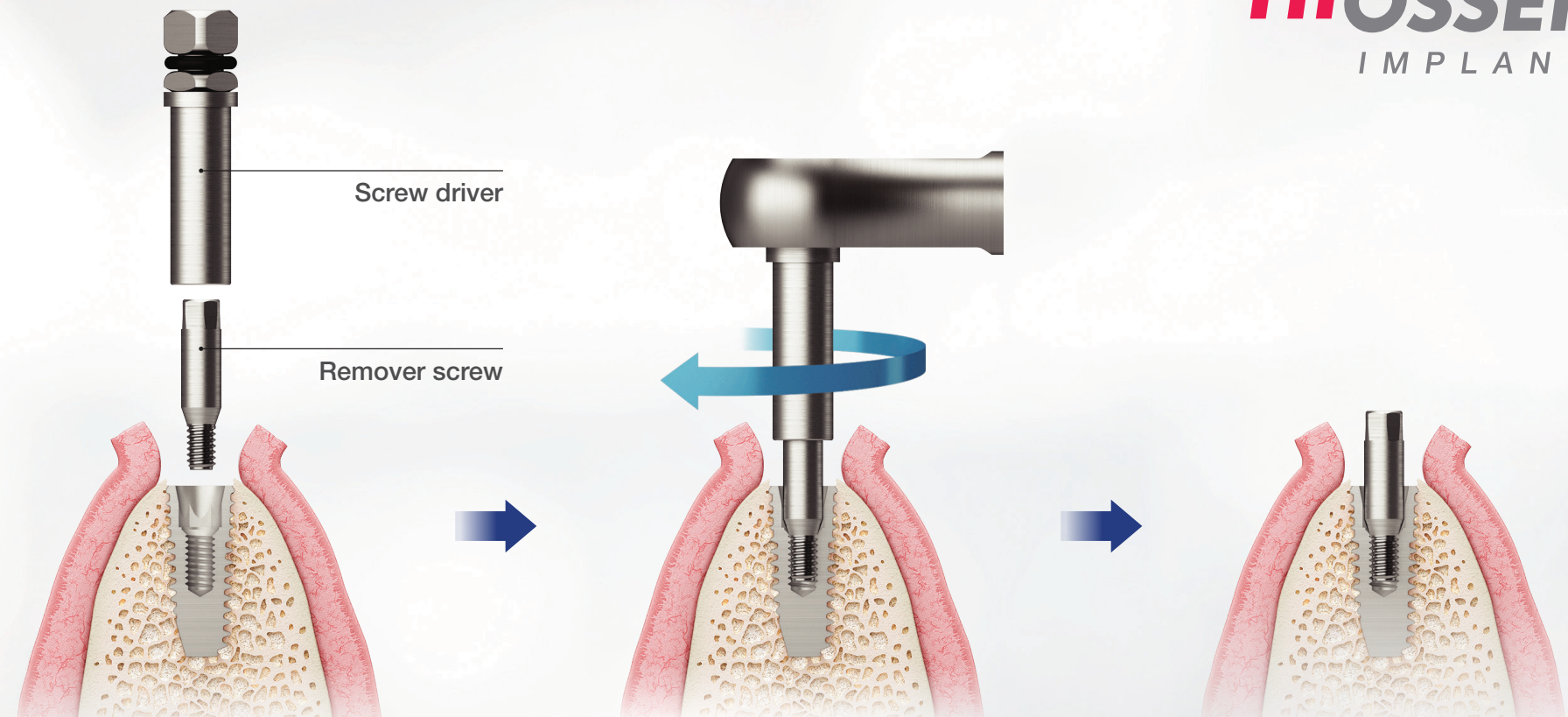

1. Select remover screw  
(Refer to the remover screw selecting guide)

2. Clockwise tightening by torque wrench  
through the screw driver  
(mini 80Ncm, regular 100Ncm)

3. Complete the removal screw  
installation into the fixture

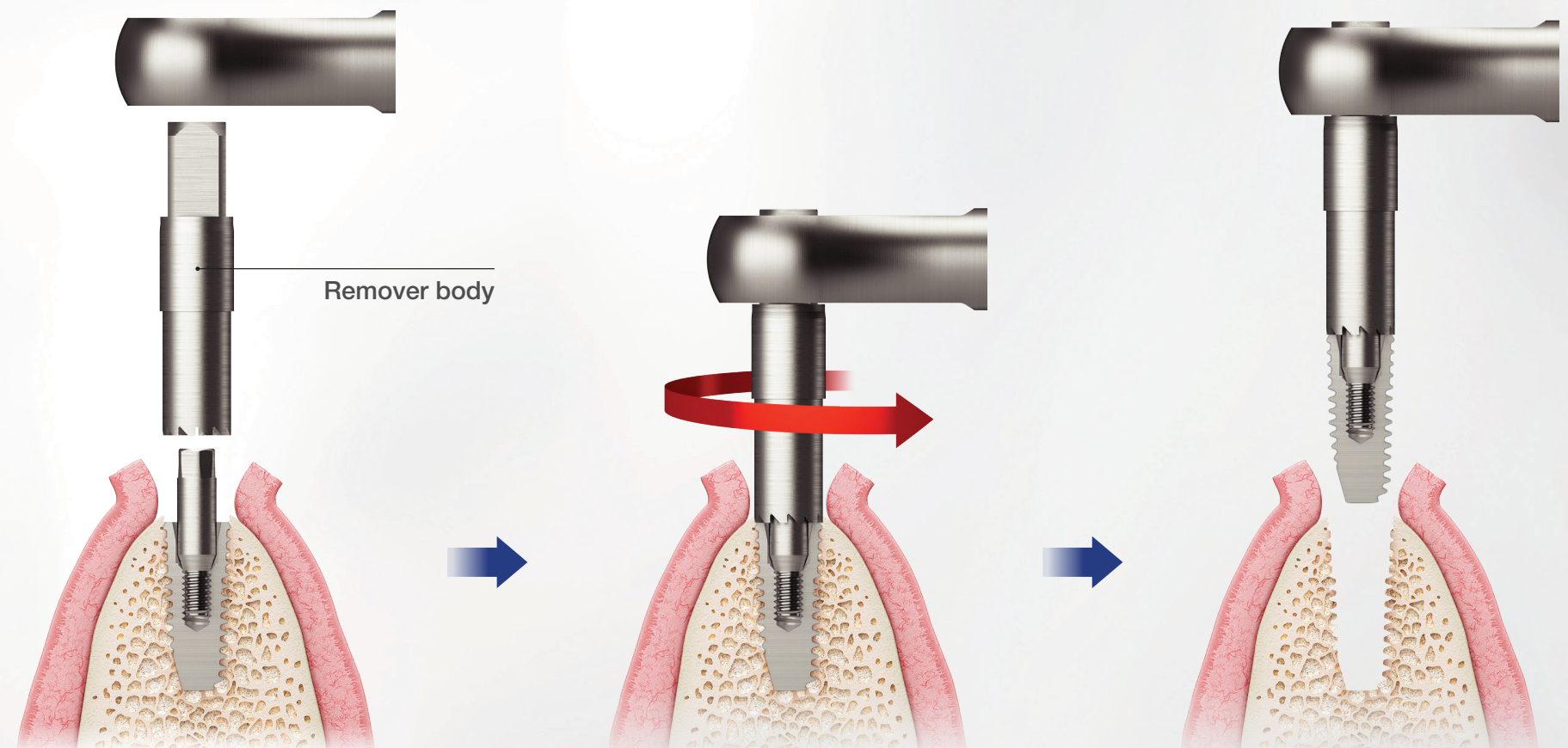

4. Removal body selection

5. Anti-clockwise screwing by torque wrench  
through the removal body  
(Maxium 400Ncm engagement possible)

6. Bone loss-less fixture removal by  
strong adhesion property between fixture  
& removal body

### Remover Screw Selection Guide

Fixture Condition Check	Normal	Crack	Partial fracture	Fracture to internal hex connection area (Only internal screw thread remained)
Remover Screw Selection	<p>Normal Remover screw connection</p>	<p>Crack Remover screw connection</p>	<p>Partial fracture Remover screw connection</p>	<p>Fracture Remover screw connection</p>